

# WIDE-EYE™

## POLYPTRAP

*“Install, Detect and Collect”*



# WIDE-EYE™

## POLYPTRAP

**Cat. No. APT013 | Wide-Eye™ Polyptrap | 25 Units/Case**

Simple Installation, Unparalleled Vision and Quick Collection means the Wide-Eye™ Polyptrap is the most efficient way to retrieve resected polyps

### FEATURES AND BENEFITS

- ✓ **Easy in-line installation**  
Installs in just seconds to minimise loss of suction.
- ✓ **Unobstructed vision**  
Unique design allows an unobstructed view.
- ✓ **Quick polyp detection**  
The white receptacle gives a contrast to help differentiate polyp from effluent.
- ✓ **Magnification**  
The inbuilt magnifying lens enables the smallest polyps to be identified quickly.
- ✓ **Simple polyp retrieval**  
Just pull the lid tab and retrieve the polyp.
- ✓ **Cost effective**  
Simple and secure polyp retrieval at a cost effective price.
- ✓ **Individually packed**  
The Wide-Eye™ Polyptrap comes individually packed to reduce the risk of contamination.

### HOW TO USE THE WIDE-EYE™ POLYPTRAP

- For detailed instructions please read the Instructions for Use, found on our website and within the Wide Eye™ Polyptrap packaging.
- When a polyp has been identified and is about to be resected, remove Wide-Eye™ Polyptrap from packaging.
- Disconnect the suction tube from the endoscope's suction barb and connect to the Wide-Eye™ Polyptrap device outflow suction port.
- Connect the free tubing from the base of the Wide-Eye™ Polyptrap to the endoscope's suction barb.
- The Wide-Eye™ Polyptrap is now ready to capture the excised polyp.
- Once the polyp is captured and visible in the trap, remove the Wide-Eye™ Polyptrap and reattach the suction tube to the colonoscope suction barb.
- Remove the lid of the trap, take out the polyp and place in a formalin pot. Rinse the trap in water and replace the lid so it's ready to reuse if a further polyp is identified in the same patient.
- Write clearly on the pathology form clinical information that relates to the contents of the formalin pot used.