

# WIDE-EYE™

## POLYP TRAP



## Efficiently retrieving resected polyps

Simple installation, unparalleled vision and quick collection means the Wide-Eye™ is the most efficient way to retrieve resected polyps.

- ✓ Easy in-line installation
- ✓ Unobstructed vision
- ✓ Magnification
- ✓ Quick polyp retrieval

## FEATURES & BENEFITS

- ✔

**Easy in-line installation**  
Installs in just seconds to minimise loss of suction.
- ✔

**Unobstructed vision**  
Unique design allows an unobstructed view.
- ✔

**Simple polyp retrieval**  
Just pull the lid tab and retrieve the polyp.
- ✔


**Quick polyp detection**  
The white receptacle gives a contrast to help differentiate polyp from effluent.
- ✔

**Magnification**  
The inbuilt magnifying lens enables the smallest polyps to be identified quickly.
- ✔


**Cost effective**  
Simple and secure polyp retrieval at a cost effective price.

## HOW TO USE THE WIDE-EYE™ POLYPTRAP


- 1**

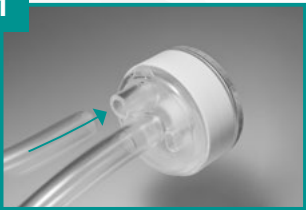


Connect the suction tube from the endoscope's suction barb to the Wide-Eye™ outflow port.
- 2**




Connect the Wide-Eye™ tubing back to the endoscope's suction barb. You are now ready to capture excised polyps.
- 3**



Once the polyp is captured in the chamber and has been inspected using the magnifier, disconnect the Wide-Eye™.
- 4**

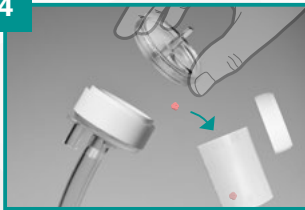

Remove the lid, take out the polyp and place in a formalin pot. Rinse the trap and replace the lid ready to reuse with the same patient.

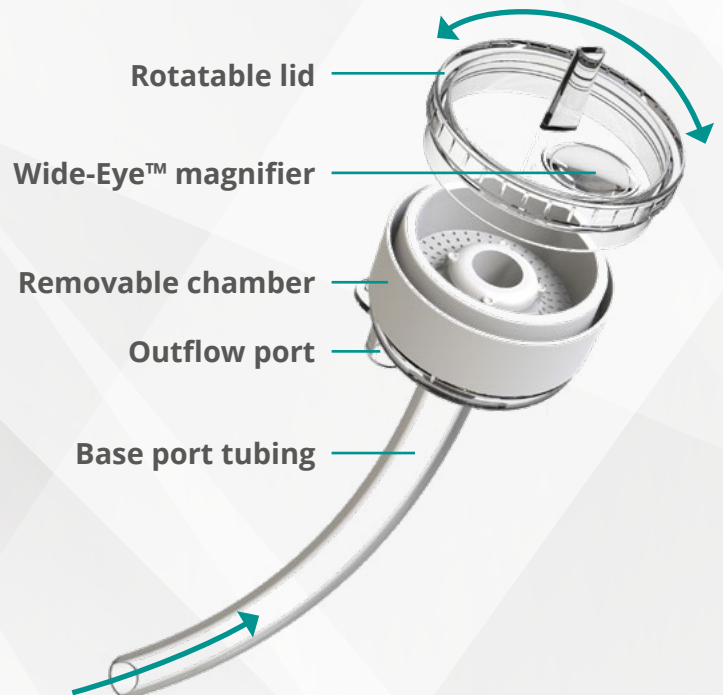

CAT. NO.	DESCRIPTION	UNITS/BOX
APT013	Wide-Eye™ Single Chamber	25

Wide-Eye™ Polyptrap is an individually packed, single use device. For detailed instructions please refer to the IFU on our website. Wide-Eye - Reference: G003001A

Arc Medical Design Ltd  
43 Park Place, Leeds  
LS1 2RY, United Kingdom  
Tel: +44 (0)113 242 0131  
Email: info@arcmedicaldesign.com



Wide-Eye™ is a trademark of Arc Medical Design Ltd.

[WWW.ARCMEDICALDESIGN.COM](http://WWW.ARCMEDICALDESIGN.COM)